

PADDED PELVIC HOLDER WITH PELVIC RESTRAINT (cod. 683P/M)

Article Name

PADDED PELVIC HOLDER WITH PELVIC RESTRAINT

(cod. 683P/M)

REF	Measure	Colour
683P/M	195	Black

Length: 195 cm

Fit: until 180 cm



Intended use

The belt is indicated for patients in the wheelchair to prevent them from falling or sliding forward of the pelvis. It is adjustable through the three plastic buckles with which the tie rods are equipped.

The medical device in question is not suitable for protection against risks not shown on this documentation, therefore we decline all responsibility for any damages and consequences that may derive from improper use.

Description and Materials

MTP: velveteen 100% polyamide or polyester (as per composition label), internal 100% ether-based polyurethane.

Gallon: 100% cotton.

Buckle: trident in acetyl plastic with resistance up to 100 kg.

Instruction for Use

- Put the harness on the wheelchair seat or seat passing the chevron or under the seat or between the seat and the backrest.
- Make the patient sit on the wheelchair or seat and adjust on the back through the plastic buckle with which the tie rods are equipped.
The device is worn over clothing.

Contraindications and Warnings

No contraindications unless reported by the attending physician.

Non-sterile device. Follow the washing instructions. Don't use if damaged.



Non-sterile device.

Maintenance and Cleaning / Washing

Periodic maintenance of the medical device consists of washing and drying operations.

Wash in warm water at a maximum of 30°C with detergent for delicate garments, rinse thoroughly but remove the water by compressing the garment into a terry towel, do not wring and dry away from heat sources.

Allow to dry horizontally; do not iron.



Wash at or below 30° C



Do not bleach



Do not tumble dry



Do not iron



Do not dry clean

**PADDED PELVIC HOLDER WITH PELVIC
RESTRAINT
(cod. 683P/M)**

Storage



In order to avoid contamination, the device should be stored in a clean, dry place (to avoid the development of mold), away from sources of direct light and heat, protected by a casing. Maintaining its physical integrity (absence of holes, cuts, fraying, unseams, wear, mold, etc.) determines the lifespan of the device.

Type of Package

Packed in plastic bag.

Disposal

Dispose the device in compliance with the regulations in force regarding environmental protection and separate collection which must be carried out in compliance with the national and local regulations in force. Check the provisions of your Municipality.

Separate waste collection:

Informative technical sheet → Paper garbage can

Plastic bag → Plastic garbage can

Pelvic Safety belt → Generic waste

Warranty

The supervision is guaranteed in accordance with the law. For manufacturing defects, consumable parts are excluded.

Other information

CND code ¹: Y122406

EMDN code: Y122406

BD/RDM code ²: 2776676



Class I medical device
Regulation (UE) 2017/745



Albo Land srl

Via Mastroianni, 9 - 40013 Castel Maggiore (Bologna) - Italy

Tel: +39 051 373838 Fax: +39 051 19875902 - www.alboland.com / info@alboland.it

In the event of a serious accident, promptly inform the Competent Authority of the Member State in which the user and/or patient is established as well as the manufacturer Albo Land srl at the following addresses:

e-mail: info@alboland.it - Tel.: +39 051 373838 / Fax: +39 051 19875902

¹ Italian Device Classification Number

² Italian Ministry of Health Registration Number
STK-683P/M (EN)